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INSTRUCTIONS FOR USE

Sample Collection Kit for DNA analysis from a blood drop

The medical device complies with the requirements of Regulation 2017/746 on in vitro Diagnostic Medical Devices (IVDR).

Manufacturer:

DNAmeter s.r.o.
U Pergamenky 1522/2
170 00 Prague 7
Czech Republic

support@dnameter.com

- ▶ **The Sample Collection Kit is intended for (self)withdrawal of blood drops from fingertip, packaging and transport of blood sample to the laboratory for analysis of DNA.**
- ▶ The Sample Collection Kit is intended for use by adults.
- ▶ A child's blood sample from a fingertip can be collected by their legal representative. **WARNING: The Lancet Device is intended for blood sample collection from children aged 8 years and older!**
- ▶ The user will collect the blood sample from fingertip by using the components from the Sample Collection Kit.
- ▶ This is not a standalone examination or a home self-test. The collection kit is intended for collecting biological material for further processing.

Reporting of Serious Adverse Events

Any Serious Adverse Event that is related to the Medical Device must be reported to Manufacturer and to the competent Authority of the Member State (<https://www.sukl.cz>).

Safety information

Please read carefully the Safety Information below before the Sample (self)withdrawal:

- ▶ Do not use if the packaging is damaged and read the Instructions for Use.
- ▶ Do not use after the Expiration Date. The Expiration Date can be found on the packaging.
- ▶ All components of the Sample Collection Kit are intended for single use only.
- ▶ Always keep the Sample Collection Kit out of the reach of children and animals.
- ▶ Consult your physician before using the test if you have a blood clotting disorder, are pregnant, or breastfeeding.
- ▶ **Do not touch the marked circles for blood sample placement inside the Collection Card.**
- ▶ The Collection Card does not contain mixtures classified as hazardous according to Regulation (EC) No 1272/2008.
- ▶ Used, unused, or otherwise unsuitable Collection Card can be disposed of in mixed waste.
- ▶ Do not use the Lancet Device if the lancet's protective cap does not cover its tip.
- ▶ **WARNING: LANCET DEVICE IS INTENDED FOR SINGLE USE ONLY! Use a second, replacement Lancet Device if needed.**
- ▶ Lancet Device sterilized using irradiation.
- ▶ When handling the Lancet Device, be careful not to activate the blade by pressing the top button of the Lancet Device.
- ▶ Remove the protective cap that covers the blade by pulling it upwards.
- ▶ The Lancet Device bottom tip, which causes a puncture and subsequent slight bleeding at the puncture site, is activated by pressing the button on the top of the Lancet Device.
- ▶ Dispose of used, unused, or otherwise unsuitable Lancet Device as hazardous waste, under code 18 01 01, according to applicable legislation and regulations.
- ▶ If you experience persistent bleeding, swelling, or signs of infection after using the Lancet Device, seek medical assistance.
- ▶ Lancet Device is generally safe when used correctly. It is important to exercise caution and follow the Instructions for Use. If you have any concerns or questions regarding the use of the Lancet Device, consult with your doctor.

The Instructions for Use, along with the Instruction for Use of Lancet Device, is available at:

<https://dnameter.com/safety>

Index of Symbols

	Read the Instructions for Use
	Single use only
	Do not use if the packaging is damaged and read the Instructions for Use
	Protect from moisture
	Protect from sunlight
	Storage temperature
	Use by Date
	LOT Number
	In vitro Diagnostic Medical Device
	Medical Device unique identifier
	CE mark Declaration of Conformity
	Manufacturer

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Package contents:

- Collection Card – 1 pc
 - Lancet Device – 2 pcs
 - Bag – 1 pc
 - Pencil – 1 pc
 - Instructions for Use – 1 pc
 - Envelope – 1 pc
- In case of sending the Sample by the shipping company.

Procedure for Sample Collection

Maintain increased hygiene throughout the using of the Sample Collection Kit. Before and after the Sample collection, wash your hands with soap and let your fingers warm up under a stream of warm water for a while. The ring finger or middle finger is suitable for sampling.

- 1. Video instructions describing the process of Sample collection are available on the Portal, where you create your User Account and find your results after the analysis is completed.**
Go to: <https://dnameter.com/start>
Before collecting the Sample, complete the first step – register yourself!
- Keep the box for later – it contains the identification number of your Sample and will also serve for its storage.
- Use the provided pencil to record the Sample Collection Date.
- For better blood flow stimulation, shortly shake your hand or massage your finger by moving it toward the fingertip.
- 5. WARNING: LANCET DEVICE IS INTENDED FOR SINGLE USE ONLY! Use a second, replacement Lancet Device if needed.**
Do not touch the button on the top of the Lancet Device which applies the puncture. First, you must remove the protective cap on the bottom side of the Lancet Device by pulling it upwards.
Press the Lancet Device onto the fingertip and press the button to activate the Lancet Device to puncture the skin to produce a blood drop.
6. Apply a drop of blood onto the Collection Card, allowing the blood to soak freely (do not touch the Collection Card with the finger). The blood must be applied by repeatedly bringing the finger with the blood drop close until both marked circles are filled to the edge. Massaging the finger helps in the formation of a blood drop.
WARNING: If you do not fill both marked circles, we may not be able to analyze your sample due to insufficient provided DNA content.
- Let the blood Sample placed on the Collection Card dry freely for at least 2 hours at room temperature. Do not accelerate drying by exposing it to heat (do not use, for example, a hairdryer or place it on a heater). Do not expose to freezing (do not place it in the freezer).
- Insert the dry Collection Card into the provided bag, then place the bag back into the Sample Collection Kit's box. Close the box and **store it at a room temperature of 15–25 °C.**
- 9. Hand over your Sample within 48 hours - follow the Instructions on the Portal (<https://dnameter.com/start>).**
Return your Sample into the DNAmeter Kiosk packaged in the plastic bag. You do not need the paper box.
If you are sending the Sample via a shipping company, place the sample in a plastic bag inside an envelope and arrange the shipment.

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Date of release of Instructions for Use: 7 April 2025